**X-RAY TUBES  
PRODUCTION**

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**TECHNICAL DATA**

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**GENERAL INFORMATIONS**

When mounting tube inserts adopt proper caution, in order to avoid glass bulb breaking and fragments projection. Please use protective gloves and glasses. Tube insert connected to H.V. supply is a radiation source: be sure to take all necessary safety cautions

- > Wash thoroughly with alcohol the external surface of tube insert (care of fire risk). Avoid contact of dirty surfaces with cleaned tube insert.
- > Clamp system inside housing or self-contained units must not mechanically stress the tube.
- > After installation, check the right working of the tube (no fluctuation of tube current nor crackling)
- > Comply with insert thermal parameters, planning and programming the exposure parameters and cooling pauses. Housing or self-contained units must be provided with an adequate thermic protection.
- > Voltages indicated in charts are valid for transformer supplied with ground center.
- > It is extremely important to observe the connection diagram and the grid resistor value. Any change could modify the dimensions of the focal spot, also varying diagnostic performances or overloading anode target.
- > Tube inserts contain environment polluting materials, particularly lead liner tubes. Please apply to qualified operator for waste disposal, according to local regulation requirements.

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