**X-RAY TUBES
PRODUCTION**

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GENERAL INFORMATIONS

When mounting tube inserts adopt proper caution, in order to avoid glass bulb breaking and fragments projection. Please use protective gloves and glasses. Tube insert connected to H.V. supply is a radiation source: be sure to take all necessary safety cautions

> Wash thoroughly with alcohol the external surface of tube insert (care of fire risk). Avoid contact of dirty surfaces with cleaned tube insert.

> Clamp system inside housing or self-contained units must not mechanically stress the tube.

> After installation, check the right working of the tube (no fluctuation of tube current nor crackling)

> Comply with insert thermal parameters, planning and programming the exposure parameters and cooling pauses. Housing or self-contained units must be provided with an adequate thermic protection.

> Voltages indicated in charts are valid for transformer supplied with ground center.

> It is extremely important to observe the connection diagram and the grid resistor value. Any change could modify the dimensions of the focal spot, also varying diagnostic performances or overloading anode target.

> Tube inserts contain environment polluting materials, particularly lead liner tubes. Please apply to qualified operator for waste disposal, according to local regulation requirements.

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


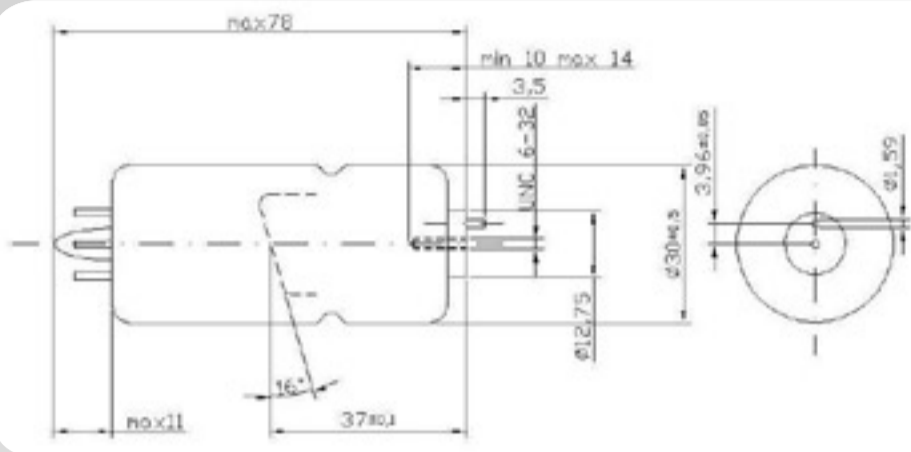
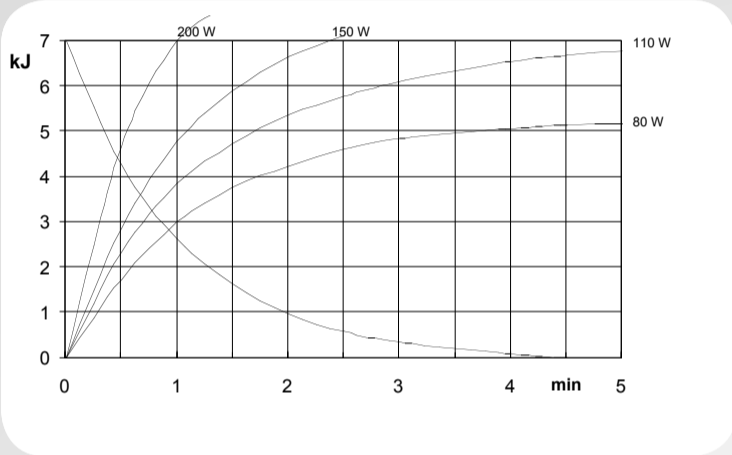
COLLEGAMENTI / CONNECTIONS / CONNEXIONS

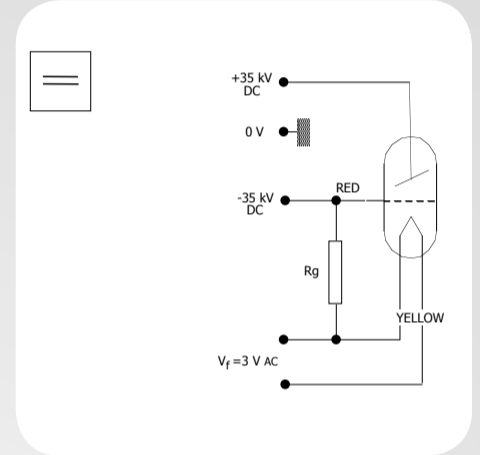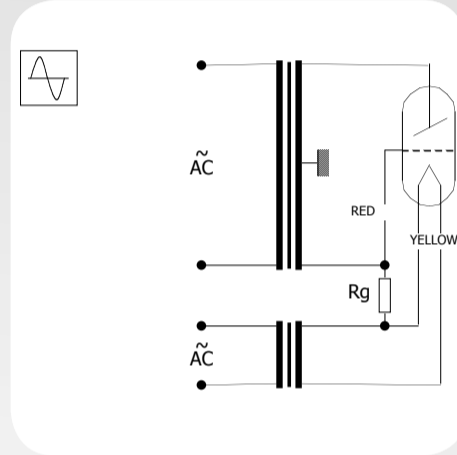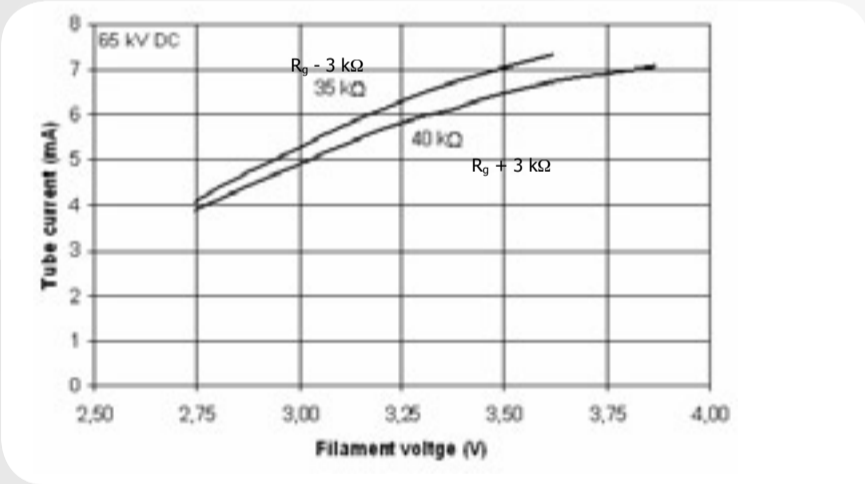**CARATTERISTICHE DI EMISSIONE E DI FILAMENTO
FILAMENT AND EMISSION CHARACTERISTICS
CARACTÉRISTIQUES D'ÉMISSION ET DU FILAMENT**

DIAGRAMMI DI CARICO / RATING CHARTS / ABAQUES DE CHARGE